


DISPOSABLE STRAPS

ECONOMY DISPOSABLE POLYPROPYLENE STRAPS

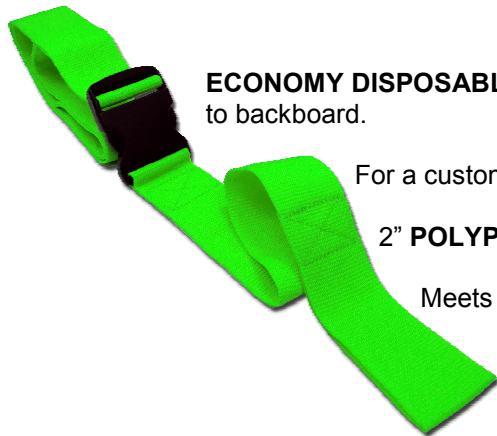


ECONOMY DISPOSABLE POLYPROPYLENE backboard straps are made to secure patients to backboard.

For a custom fit, economy disposable poly straps are easily adjustable.

2" **POLYPROPYLENE** webbing (1800 pound break strength)

Meets FDA requirements.

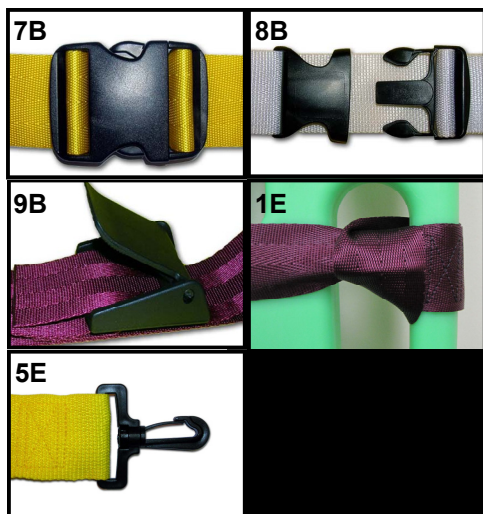

INSTRUCTIONS:

- 1.) Inspect strap before use.
- 2.) Position patient.
- 3.) Strap should have proper tension with no twists.
- 4.) Patient should be flat on backboard/stretcher.
- 5.) Wrap strap around patient and backboard/stretcher.
- 6.) Buckle and tighten strap to secure.

Intended as a guide only.
 Consult your local medical agency for all protocols.

7B Plastic Side Release Buckle 400 Pound Break Strength	1E Loop Lock Ends
8B Plastic Double Adjust Buckle 400 Pound Break Strength	5E Plastic Swivel Speed Clip Ends 140 Pound Break Strength
9B Plastic Cam Buckle 5000 Pound Break Strength	

Black	Orange	Neon Green	Red
Blue	Green	Yellow	Olive Drab
White			

